

PROTECTIVE BARRIER FOR FIRST RESPONDERS

BENEFITS

- + Waterproof, permeability tested
- + Transparent, allows visualization of site
- + Very quick and easy to use when time is of the essence
- + Keeps a wound site free of bacteria and allergens
- + Self-adhering, will not stick to skin or hair

HOW TO USE

- + Apply SealSkin Wrap 2 to 3 inches above and below site to ensure a watertight seal
- + While wrapping, keep SealSkin taut to provide better control and coverage
- + Use scissors or your thumb to tear the wrap off at the desired length and seal the end in place with your hand

TO REMOVE

- + Use scissors, unravel or tear wrap
- + Always assess the integrity of the site after removal

USES

- + Can secure an IV and/or endotracheal tube in place during transport
- + Helps to stabilize bleeding by creating compression over a dressing
- + Immobilize fractures quickly by wrapping a splint with SealSkin
- + Will secure in place dressings, bandages and ice packs
- + Secure IV lines



Hospital Tested and Approved
Passed Bio-Compatibility Testing
FDA Registered
Veteran Owned/WBENC Certified

