



A WATERPROOF, PROTECTIVE BARRIER THAT HELPS THE HEALING PROCESS FOR ASSISTED LIVING AND LTC PATIENTS

BENEFITS

- + Waterproof, permeability tested
- + Self-adhering, will not stick to skin or hair
- + Gentle on fragile, sensitive skin
- + Transparent, allows visualization of site
- + Allows patients to bathe while keeping dressings dry and secure

HOW TO USE

- + Apply SealSkin Wrap 2 to 3 inches above and below site to ensure a watertight seal
- + While wrapping, keep SealSkin taut to provide better control and coverage
- + Use scissors or your thumb to tear the wrap off at the desired length and seal the end in place with your hand

TO REMOVE

- + Use scissors, unravel or tear wrap
- + Always assess the integrity of the site after removal

USES

- + Protective wrap that will provide a barrier to keep things clean, dry and help prevent infection
- + Easily secures dressings, ice packs and bandages in place



Hospital Tested and Approved
Passed Bio-Compatibility Testing
FDA Registered
Veteran Owned/WBENC Certified



Made in USA

LEARN MORE AT [SEALSKINWRAP.COM](https://www.sealskinwrap.com) OR CALL (980) 446-2111